

## EVOLUTION SEATED SCOOTER

Print

Ready to purchase your KneeRover?

### Product Features

- Padded seat allows patient to rest in comfort while

BUY NOW!

elevating the non-weight bearing leg on

front foot peg compactly for easy storage and

Quick-release basket included

rubber wheels with sealed bearings; indoor or outdoor use

movement with no hopping or

- Hand Grips for added comfort
- Adjustable Disc Hand Brake
- Color = Blue

BUY NOW!

towards the ground to lock the steering column in the upright position (rotate the lever counter clockwise to loosen and clockwise to tighten the pressure on the lever).

To lower the steering column, first pull the lever up to release the clamp and unlock the steering column. Then simultaneously push the lever to the left and downward to disengage the pin and gently guide the steering column downward.

Interested in purchasing one of our KneeRover products? Please visit our exclusive online partner:

**Knee Walker Shop.com**

free to move

- Folds for transport
- Large 8-inch great for
- Smooth skipping



DESCRIPTION

SPECIFICATIONS

TESTIMONIALS

USER MANUAL

PRODUCT INQUIRY

### PRODUCT DESCRIPTION

The Evolution Mobility Scooter Utilize this device for maximum mobility during your recovery from a full range of foot, ankle, knee, leg, or hip injuries. The versatile Evolution Mobility Scooter gives the patient the benefit of a scooter in the seated position Ideal solution for patients who cannot bear weight on the foot/ankle, knee or hip due to an injury or illness. The large gel padded seat and convenient front leg peg allows the patient to rest in comfort while elevating the non-weight bearing leg. It can be a great long or short term alternative to crutches or folding walkers that can have adverse affects on the back or under arm pain from extended use.

Operations Steering Column Setup Raise the steering column and simultaneously push the lever at the base of the steering column to the left and downward to disengage the pin. Push the steering column upward until it is in the full upright position. Release the lever to engage the safety pin. Pull the lever downward so the end of the lever is pointing

Testimonials

I highly recommend using a scooter to anyone with a fractured ankle. No more worries of falling, no muscle fatigue from jumping around on one foot. You can drive one handed and carry a cup of coffee! I can get [...]

- Kittyd

Still Have Questions?




FAQ

find answers fast



Happy Customers

read customer testimonials

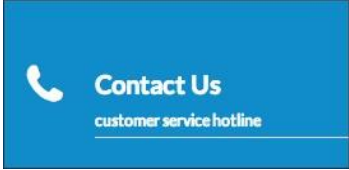

### Handle Bar Height Adjustment

The best position for the handlebar is approximately hip-level. You can raise/lower handlebars by loosening the screw knob on the steering column, adjusting handlebar height, and retightening the screw knob.

### Seat Setup and Adjustment

The Evolution Mobility Scooter can be used for right or left leg injuries. To setup the seat and/or adjust the height, remove the scooter pin and place the seat at the desired height then reinsert the pin. Pull the lever downward so the end of the lever is pointing towards the ground to lock the seat in the desired position (rotate the lever counter clockwise to loosen and clockwise to tighten the pressure on the lever).

### Operation

- Place hands on handlebars
- While seated place heel of injured leg on foot peg
- Release parking brake and propel yourself forward or backward with your pushing foot

DESCRIPTION      SPECIFICATIONS      TESTIMONIALS      USER MANUAL      PRODUCT INQUIRY

<b>FEATURES &amp; SPECS</b>	
<b>EVOLUTION SEATED SCOOTER</b>	
<b>GENERAL</b>	
Weight ( bs)	
WeightCapacity ( bs)	
Recommended User Height Range 4'11"•t06'11"	
Terrain Use	Indoor & Some Outdoor
Steerable	YES
Folds for Easy Transport	YES
Color	Blue
<b>DIMENSIONS</b>	
Overall Length (inches)	335
Width (inches)	15.75
Front Axle Width (inches)	15.75
Folded Height(inches)	
<b>KNEE PLATFORM (DISTANCE FROM GROUND TO TOP OF KNEE PAD'</b>	
Lowest Position ( inches)	2 (Seat)

Highest Position (inches)	31 (Seat)
<b>KNEE PAD</b>	
Width (inches)	N/A
Length (inches)	N/A
Thickness (inches)	N/A
<b>HANDLE BARS</b>	
Min Height(inches)	
Max Height(inches)	
Grips	Ergo Dynamic Easy Clean Rubber
<b>WHEELS</b>	
Diameter (inches)	7.5
Number	
Type	Solid Non Marking Rubber
<b>BRAKE</b>	
Type	Rear Disc Brake
Parking Brake	YES
<b>BASKET (INCLUDED)</b>	
Dimensions (inches)	125x8x8
Detachable	YES
<b>ASSEMBLY</b>	
Tool Free Setup	YES

Interested in purchasing one of our KneeRover products? Please visit our exclusive online partner:

**Knee Walker Shop.com**

free to move

**CUSTOMERSERVICE**

Our exclusive online partner KneeWalkerShop.com offers a convenient toll free Customer Service Hotline, open

Monday -Friday 9 am to 5 pm EST.

**1-877-737-9877**

Interested in more information, click here to fill out our form:

**CONTACT US**

**ABOUT KNEEROVER**

We are committed to providing innovative and affordable mobility solutions to get you moving as soon as possible after injury or surgery and for the duration of your recovery - free to go about your busy, active life - and have a little fun doing it. We are all about leaving the painful crutches behind and giving you the Freedom to Move.

**CHOOSE YOUR KNEEROVER**

Not sure which KneeRover product is right for you? We can help!

**FIND OUT MORE**

**BUY NOW!**