

3WHE

STEEL

Rollator



Getting Around
Has Never
Been Easier...

INSTRUCTIONS FOR ASSEMBLY

WARNING! It is very important that you read all instructions before assembly and use of this rollator.

Before using the rollator, be sure your weight is within the weight capacity of the unit. This rollator requires little assembly and can be performed without tools.

PARTS OF ASSEMBLY:

2 Handlebars with Loop-Lock Brakes; 1 Snap Pouch

ASSEMBLY INSTRUCTIONS:

1. Open your rollator. Push the folding hinge down until it locks securely in the straight position.

2. Place both loose hand pieces, with the pre-assembled locking loop brakes, into the handlebar openings and adjust to the desired height. To do this, twist the height adjustment knob on the outside of each frame opening until tight.

(CAUTION: INSERT AT LEAST 2" OF HANDLEBAR INTO OPENING TO ENSURE PROPER ALIGNMENT OF THE HEIGHT ADJUSTMENT KNOB. HANDLE SHOULD NOT EXCEED 40" FROM THE GROUND.)

3. Test and adjust the tension on the braking system mechanism to ensure proper and secure braking before use.

4. Place snap pouch onto the rollator by placing the smaller side of the triangle shaped pouch to the front of the rollator. Attach the pouch by snapping the straps around the legs on both sides of the rollator.



ITEM # 501 - 1014 - 2100

SPECIFICATIONS

Overall Width 27" Folding Width 11"
Wheels 8" x 2" Pouch 13" x 8" x 11" deep
Handle Height 31" - 36.75"
Width Between Handles 17"
Weight 14.3 lbs. Weight Capacity 250 lbs.

DURO-MED | *Lifestyle Products for Better Living™*

USE OF NEW ROLLATOR

BEFORE USE, PLEASE READ ALL CAUTION AND WARNING LABELS ON THE ROLLATOR

WALKING

This unit is to be used as a mobility aid creating a natural balance. With hands grasped securely onto handles, the user walks slowly behind the rollator. Gently, push forward with weight distributed evenly between handlebars.

BRAKING

Normal Brake can be activated by pulling the brake handle up, similar to operating the brakes of a bicycle. This function may be applied, while walking with the rollator to slow down momentum, stop for a moment, or maneuver up, down or around obstacles which may be in your path.

Parking Brake can be activated by pushing the handle down into the lock position. This function should be activated when the user stops for a period of time. For safety reasons, the parking brake should always be engaged before walking away from the rollator. Also if shopping, it will be a good idea to activate the parking brake while stopped in an aisle, to avoid rollator from advancing forward. To release the parking brake, gently pull up. The braking system is convenient and easy to operate. People who may have hand disabilities should find the brake handles wide, comfortable and non-binding. Minimum pressure will need to be applied to activate the locking loop brake system.

FOLDING

Pouch can remain on rollator while being folded. From the user side, release the lock mechanism located on the right of the rollator by pulling toward the user. Grasp handlebars and gently squeeze together until folded. The rollator folds easily for storage or transport.

MAINTENANCE

Always make sure your brakes are operating correctly before use. Keep the brake handles free from dirt, dust and grime. Build-up of particles could cause your brakes not to engage properly. If the brakes are not braking, check the cables for looseness. The cables can be tightened at the adjuster screw located where the cable is connected to the brake handle. If the cable is broken, it will need to be replaced. Periodically check and tighten all nuts,



WALKING MADE EASY



SQUEEZE TO



RELEASE TO



PUSH DOWN TO LOCK



bolts, screws, and other hardware located on **RELEASE LOCK**
AND rollator. **PULL BACK INTO**

FOLDING POSITION

www.duomed.com

LIMITATIONS AND EXCLUSIONS

Products which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond reasonable control are not covered by this warranty.

Parts or materials which are subject to normal wear resulting from the use of this product, and which must be replaced or repaired due to such normal wear, are the owner's responsibility and are not covered by this warranty.

This warranty is exclusive and is in lieu of all other express warranties, implied warranty, including but not limited to the implied warranty of merchantability and fitness for a particular purpose, and shall not extend beyond the duration of the express warranty provided herein. DMI will not be liable for consequential or incidental damages whatsoever.

This warranty gives you specific legal rights which may vary from state to state. Some states do not allow the exclusions or limitations of incidental or consequential damages, or how long an implied warranty lasts, so the exclusions and limitations may not apply to you.

WARNING! READ BEFORE USING

This unit is not intended to be used as a wheelchair. At regular intervals, check and tighten all fasteners and hinged parts. Check all plastic parts for wear often. Units are not designed to assist users from a sitting to a standing position. When stopping, do not lean excessively on the unit. The brakes are not designed to withstand excessive pressure. Excessive use causes the brake cables to stretch or fray and adjustment or replacement of the brake cable may be required. DMI rollators are not suggested for people with severely limited ambulation for whom a rigid walker may be recommended. Please consult with your physician before using this

product especially if you are taking any medication. Failure to observe this warning may result in injury.

**CAUTION! CHECK BRAKES BEFORE
USING**

Always check your brakes before using your rollator. Keep your brake handles in excellent condition at all times. Always check for loose parts, tightness of the cables and cleanliness.

It is best to keep your rollator fully assembled at all times.

Distributed by:
Duro-Med Industries
Div. of DMS Holdings,
Inc.
1931 Norman Drive South
Waukegan, IL 60085 USA
www.duromed.com

Made in China
Toll-Free Customer Care Help
Line
877-505-6190
#91-032-1014 02/07
©2007 Duro-Med Industries