


vive®

# QUAD CANE

User Manual

CAN1003

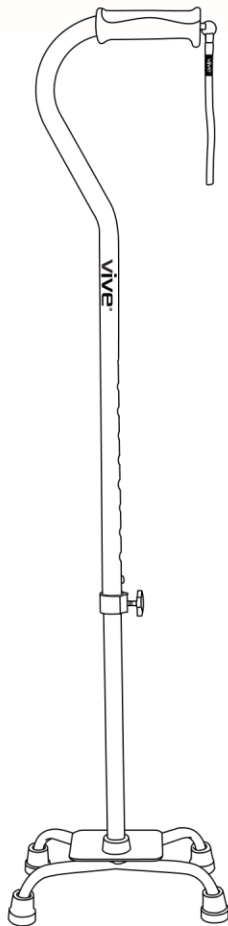

[vivehealth.com](http://vivehealth.com)

## PRODUCT DESCRIPTION

The Vive quad cane is our premiere cane for stability and support. An anodized aluminum frame allows for the cane to be lightweight while still supporting up to 300 lbs. The height is adjustable in 1" increments ranging from 28"-37". The four-prong base allows the cane to stand upright without any assistance. This solid base gives you the ability to solely focus on your stability instead of worrying about balancing your weight like you would with a single pronged cane. Protected by a lifetime guarantee.

## WHAT'S INCLUDED

- 1x Quad Cane
- Lifetime Warranty

## WARNING

- Do not install or put together cane, without first reading and understanding instructions and warnings, please contact healthcare professional to help you setup and install; otherwise an injury or accident may occur.
- Do make sure to double check rubber tips to ensure they are in good condition. Look for tips, tears, cracks, or wear. If any of these exist replace immediately. This cane can provide and assist individuals up to 300 pounds. This cane is designed to provide extra support, stability, assistance walking, and promote balance. This cane is NOT made to support the total weight of individual using the cane.
- Do make sure the snap button fully locks into place and protrudes through the adjustment hole on other side of cane. This secure th cane height into a locked position. Make sure before using cane, that it is securely locked into place; and the locking screw has been tightened all the way.
- Always consult your doctor or physical therapist to determine proper adjustments and usage.

## OPERATING INSTRUCTIONS


### Correct Cane Height

1. Stand in upright position on a flat surface. Let your arm hang down naturally by your waist.
2. Adjust cane to stop part of handle touches the inside of your wrist and palm of hand.



## Cane Height Adjustment

1. Turn locking screw to the left (counterclockwise) to loosen.
2. Press in the button snap.
3. Slide extension tube up or down to desired height. Button snap should secure cane into place.
4. Tighten locking screw to the right (clockwise) to tighten and lock cane height into place.

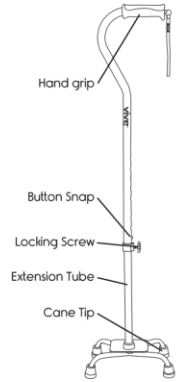

## MAINTENANCE & CARE

1. Double check button snap to make sure it releases and snaps back into place.
2. Inspect rubber cane tips for damage and wear periodically.
3. Replace any damaged, broken, or worn out parts immediately before using.

## WARRANTY REGISTRATION

This product is guaranteed to be free from manufacturing defects. We will replace or exchange any product that is defective to your complete satisfaction. This warranty extends for the lifetime of the purchaser, as long as the product has not been misused or abused.

To ensure your warranty,  
please register your product at  
[www.vivehealth.com/register](http://www.vivehealth.com/register).

Product Code: CAN1003



We sincerely appreciate your business.  
We strive to provide you with the best  
quality products at great value.

If you have any questions please contact us.



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808

Monday - Friday 9am - 5pm EST